EN Instructions for use

Purpose

BlueBec® BB200 series respirators are personal protective equipment (PPE) designed to protect against the inhalation of aerosols and dusts, in accordance with protection class FFP2, as defined in standard EN 149:2001+A1:2009. This PPE reduces the likelihood of the wearer's breathing air coming into contact with microorganisms, blood and body fluids. It can be used for infection control purposes.

The mask is not intended for purposes other than those described and must not be used in these cases.

Warnings and limits of use

Respiratory protection can only be effective if the appropriate adjustments are made to ensure tightness and the mask is worn during the entire stay in the danger zone.

This product is NOT suitable for persons with beards or other facial hair in the area where the mask is worn, as proper sealing cannot be guaranteed.

Do not use the respirator in an environment where metal can cause a hazard because the nose clip contains metal.

Leave the danger zone immediately if the quality of the air you breathe is poor, if the concentration of contamination is harmful to health or life, if breathing becomes difficult, or if pain or dizziness occurs.

Only use the mask in adequately ventilated areas and not in an oxygen content $\leq 19.5\%$ or in a potentially explosive atmosphere.

The product does not protect the user from gases or vapours.

The respirator is not suitable for protection against air pollution/concentrations that are unknown or present an immediate danger to life and health.

The product is only intended to be worn for one shift (≤ 8 h).

Dispose of and replace respiratory protection immediately if it has been contaminated with blood or other infectious materials, if it has been damaged or at the end of your shift.

This product is not reusable and must not be modified or repaired.

Masks without a valve can reduce the release of exhaled microorganisms and thus the contamination of the environment.

This product cannot eliminate the risk of contagion or infection.

Failure to follow the instructions and warnings may reduce the effectiveness of respiratory protection.

Storage

The product must be stored in a clean and dry environment within a temperature range of $-20\text{ }^{\circ}\text{C}$ to $+30\text{ }^{\circ}\text{C}$ and at a relative humidity of $< 80\%$ in the original packaging.

Explanation of the symbols

Follow the instructions for use

End of storage period
Format: YYYY/MM

NR

Not reusable

Max. temperatur range during
storage

LOT

Batch number

Max. relative humidity
during storage

Date of manufacture
Format: YYYY/MM/DD

Dispose properly
(according to local regulations)

Instructions for fitting the mask

Make sure your hands are clean before unpacking the mask.

Do not open the foil packaging until immediately before use.

Check all mask components for damage before putting it on.

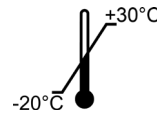
- (1) Unfold the mask and bend the nose clip until it is completely flat. Then preform the nose clip into a slight curve. Make sure the kink in the middle of the nose clip is flattened.
- (2) Now place the mask from below over your chin and nose by holding the ear loops.
- (3) Then pull the ear loops over your ears.
- (4) Now adjust the nose clip to your nose shape with both hands.
- (5) A leakage check should be carried out before entering the workplace or danger zone.

Instructions for removing the mask

To take off the mask, pull the ear loops off from back to the front over your ears. Do NOT touch the potentially contaminated mask body. Dispose of the mask properly according to local regulations.

Leakage check

1. Gently place both hands over the mask without letting it slip.
2. Breathe out strongly.
- 3a. If air leaks in the nose area, readjust the nose clip. Check for leakage again.
- 3b. If leakage occurs at the edge of the mask, adjust the fit of the mask and ear loops. Check for leakage again.
4. If a proper fit cannot be achieved even by readjustment, the mask will not function properly. In this case, do NOT enter the danger zone!

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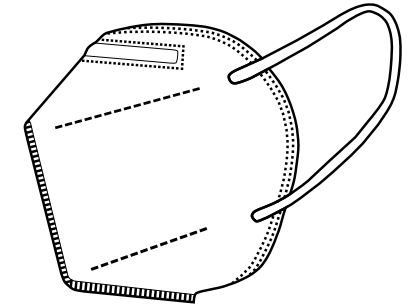
EN Instruction for use

Filtering half mask
Model series BB200

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