## EN Instructions for use

### Purpose

BlueBec® BB200 series respirators are personal protective equipment (PPE) designed to protect against the inhalation of aerosols and dusts, in accordance with protection class FFP2, as defined in standard EN 149:2001+A1:2009. This PPE reduces the likelihood of the wearer's breathing air coming into contact with microorganisms, blood and body fluids. It can be used for infection control purposes.

The mask is not intended for purposes other than those described and must not be used in these cases.

### Warnings and limits of use

Respiratory protection can only be effective if the appropriate adjustments are made to ensure tightness and the mask is worn during the entire stay in the danger zone.

This product is NOT suitable for persons with beards or other facial hair in the area where the mask is worn, as proper sealing cannot be guaranteed.

Do not use the respirator in an environment where metal can cause a hazard because the nose clip contains metal.

Leave the danger zone immediately if the quality of the air you breathe is poor, if the concentration of contamination is harmful to health or life, if breathing becomes difficult, or if pain or dizziness occurs.

Only use the mask in adequately ventilated areas and not in an oxygen content\* $\leq$  19.5% or in a potentially explosive atmosphere.

The product does not protect the user from gases or vapours.

The respirator is not suitable for protection against air pollution/concentrations that are unknown or present an immediate danger to life and health.

The product is only intended to be worn for one shift ( $\leq$  8 h).

Dispose of and replace respiratory protection immediately if it has been contaminated with blood or other infectious materials, if it has been damaged or at the end of your shift.

This product is not reusable and must not be modified or repaired.

Masks without a valve can reduce the release of exhaled microorganisms and thus the contamination of the environment.

This product cannot eliminate the risk of contagion or infection.

Failure to follow the instructions and warnings may reduce the effectiveness of respiratory protection.

### Storage

The product must be stored in a clean and dry environment within a temperature range of  $-20^{\circ}\text{C}$  to  $+50^{\circ}\text{C}$  and at a relative humidity of  $< 70\%$ .

### Explanation of the symbols

	Follow the instructions for use		End of storage period Format: YYYY/MM
<b>NR</b>	Not reusable		Max. temperature range during storage
	Batch number		Max. relative humidity during storage
	Date of manufacture Format: YYYY/MM/DD		Dispose properly (according to local regulations)

### Instructions for fitting the mask

Make sure your hands are clean before putting on the mask.

All mask components should be checked for damage before use.

- Unfold the mask. Perform the nose clip into a slight curve. Make sure the kink in the middle of the nose clip is flattened.
- Now pass the included head strap through the ear loops of the mask.
- Guide the ends, in the same orientation, through the open cord clamp and fix the ends by closing the clamp.
- Now put the mask on your chin and nose and fix it with one hand. The straps are placed on the outside of the mask.
- Take the head strap with your free hand and pull it onto the back of your head.

Position the head straps according to your personal needs to ensure a secure hold and tight fit of the mask. Possible positions of the head straps are shown in Figure 5.

Note: By pulling the ends of the straps through the cord clamp, you can adjust the pulling force of the mask to suit your individual needs. Always pull on both strap ends simultaneously.

- Use both hands to adjust the nose clip to the shape of your nose.
- A leakage check should be carried out before entering the workplace or danger zone.

Note: For proper use, the mask must be worn exclusively in conjunction with the included head strap extension.

### Instructions for removing the mask

To take off the mask, pull the elastic bands from the back to the front over your head. Do NOT touch the potentially contaminated mask body. Dispose of the mask properly according to local regulations.

### Leakage check

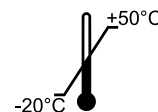
- Gently place both hands over the mask without letting it slip.
- Breathe out strongly.
- If air leaks in the nose area, readjust the nose clip. Check for leakage again.
- If leakage occurs at the edge of the mask, adjust the fit of the mask and straps. Check for leakage again.
- If a proper fit cannot be achieved even by readjustment, the mask will not function properly. In this case, do NOT enter the danger zone!

### Video

A demonstration of the correct application can be found at:

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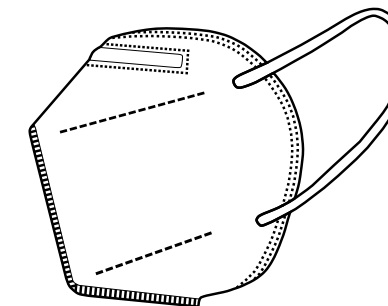
### EN Instructions for use

Filtering half mask  
Model series BB200

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